PRODUCT INFORMATION

VizOpti® Preservative-Free Irritated Eye Drops

AVAILABLE LANGUAGES

English

Dansk

Svenska

Norsk

Suomi

Íslenska

© 2018 All rights reserved. VizOpti is a registered trademark of VizOpti Ltd.

Instructions for Use

VizOpti Preservative-Free Irritated Eye Drops

Soothing relief for eyes irritated by pollen, allergies and other irritants

RELIEF

For irritated itchy eyes caused by factors such as hay fever, allergens, dust, pollution, air conditioning , smoke and bright lights

INSTANT SOOTHING AND COOLING

For irritated eyes: Relieves itching, burning grittiness and discomfort

EXPERT FORMULATION

With Natural Plant Extracts of Calendula and Hamamelis

PRESERVATIVE FREE

Ideal if you need to apply drops more frequently, have sensitive eyes or wear contact lenses

DESCRIPTION

The product is a clear solution, supplied in 10 ml /15 ml OSD Preservative free container (blue cap)

MODEL DETAILS

Formulation code: IRRITPF

INGREDIENTS

Sodium chloride 0.75%, calendula extract and hamamelis extract, disodium EDTA in a borate buffer 0.125%

DIRECTIONS FOR USE

- Ensure your hands are clean and dry before use
- Shake before use
- Check the bottle seal is not broken or missing before use. To open the bottle: Twist the cap anti-clockwise to break the seal.
- Tilt your head backwards and gently squeeze 1 to 2 drops into the eye
- Blink several times to cleanse the eye
- Repeat for the other eye as necessary.
- Replace cap after use.
- May be used prior to and following contact lens wear. May be used prior to and following contact lens wear.
- Use as often as required / safe for everyday use
- Compatible with contact lenses

WARNINGS:

- The product is sterile until first opened
- Do not use if tamper-evident seal is missing or broken.
- Do not swallow the solution
- If eye irritation occurs, discontinue use and consult an eye care practitioner. If condition worsens, consult a specialist.
- If you have been diagnosed with a medical eye condition or are using other eye medication, consult a healthcare professional before use.

- Do not use if allergic to any of the ingredients.
- May cause temporary blurring of vision, if affected do not drive or perform skilled tasks until vision is clear.
- For use in the eyes only
- This product should not be used to soak or disinfect contact lenses
- To avoid contamination, do not touch the dropper tip.
- Use within 90 days of first opening / Once opened, discard after 90 days
- Do not use after the expiry date
- For adults - May be used by children younger than 16 years under supervision

This product should not be used on children younger than 2 years without medical advice as it contains boron and may impair fertility in the future.

In the event of a serious incident occurring during the use of this medical device or as a result of its use, please report the incident to the manufacturer and the Competent Authority of the Country in which you are located

KEEP OUT OF SIGHT AND REACH OF CHILDREN.

STORAGE

Store between 5 °C and 30 °C

Avoid direct sunlight.

Do not use if package is open or damaged

East Midlands Pharma Ltd
Unit 1 Rothley Lodge, Loughborough Road
Rothley, Leicestershire, LE7 7NL, UK

Advena Ltd, Tower Business Centre,
2nd Flr, Tower Street, Swatar, BKR 4013
Malta

2797

Revision History

Rv	Date	Description
01	04 June 2026	New electronic Instructions for Use (brand specific)

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